

### The SaltFacial Glycolic Acid DermMasque

Active Ingredients:

- 30% GlyPure Glycolic Acid
- Lactic Acid
- Tetrahexadecyl Ascorbate – stable, deep-penetrating, oil-soluble Vitamin C; powerful anti-oxidant; promotes collagen synthesis; helps protect against cell damage.

Intended Use - self-neutralizing masque formulated for:

- Increased post-treatment exfoliation
- Sun damaged skin with uneven texture
- Acne and acne prone skin
- Scars and stretch marks
- Skin lightening and correcting pigment

### The SaltFacial Mandelic Acid DermMasque

Active Ingredients:

- 10% Mandelic Acid – antiseptic, anti-bacterial properties
- Lactic Acid
- Tetrahexadecyl Ascorbate

Intended Use - self-neutralizing masque formulated for:

- Sensitive skin
- Sun damaged skin and uneven texture
- Acne and acne prone skin
- Skin lightening and correcting pigmentation

### The SaltFacial Gliding Gel

Active Ingredients:

- Plant stem cells & Citrus Limonium SWT-7 + Naolys Fiber Booster Citrus

Intended Use - hydrating and soothing gel; coupling agent for ultrasound and RF treatments.

### INSTRUCTIONS FOR USE:

1. Apply approximately ½ - 1 tsp (3-5 pumps) to the treatment area – avoiding the eyes and any areas of sensitivity.
2. Gently ‘press’ into treatment area until absorbed.
3. Apply approximately 1–2 Tbs of The SaltFacial™ Gliding Gel to the treatment area. Additional Gliding Gel may be added as needed during treatment.
4. Once treatment is complete, remove excess gel with a soft, damp cloth.